


Quick Reference Guide

– Altair

Description

The Human Care Altair overhead lift is a lightweight portable lift with a discreet and modern design. Its portability ensures that asset investment is greatly reduced as the same lift easily can be used in multiple rooms. The Altair features an advanced technology platform that allows tracking of lift usage and visualization of collected data. Altair features a unique telescoping hanger bar that offers different widths in one product to fit any patient and resulting in a comfortable sitting position. The lift is available in three weight capacities, 150kg, 220kg and 300kg, and can be used with either a 2, 3 or 4-point suspension.

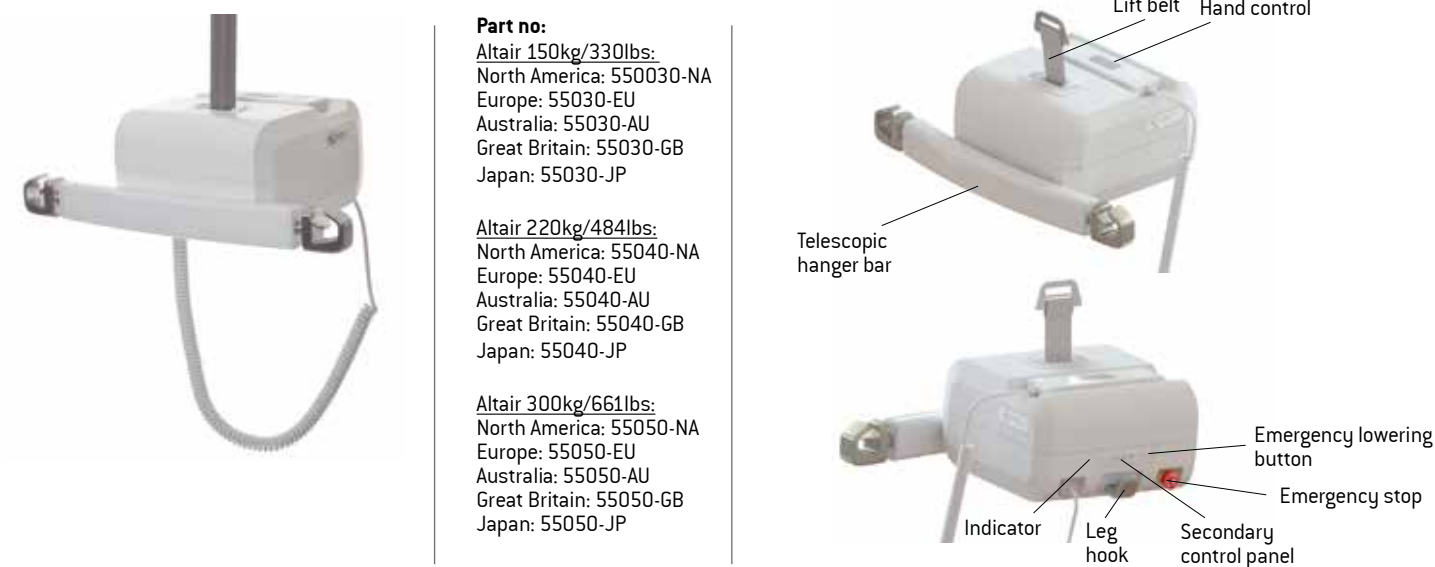

Part no:
Altair 150kg/330lbs:
 North America: 550030-NA
 Europe: 55030-EU
 Australia: 55030-AU
 Great Britain: 55030-GB
 Japan: 55030-JP

Altair 220kg/484lbs:
 North America: 55040-NA
 Europe: 55040-EU
 Australia: 55040-AU
 Great Britain: 55040-GB
 Japan: 55040-JP

Altair 300kg/661lbs:
 North America: 55050-NA
 Europe: 55050-EU
 Australia: 55050-AU
 Great Britain: 55050-GB
 Japan: 55050-JP

Using the lift

Select and position sling around the patient according to sling instructions. Use hand control to position carry bar directly above the patient.	Attach the head sling loops to carry bar first, than leg straps to leg hooks.	Raise the patient (up on hand control) and as raises check to ensure straps are firmly in place.	Raise the patient up until obstacles are clear, hold sling and position/move patient as needed.	Lower patient (down on hand control) until in desired position. Continue to lower until all straps are slack to remove


Helpful Troubleshooting Tips!

The lift will not work while plugged in and/or if emergency stop is engaged

Quick Reference Guide


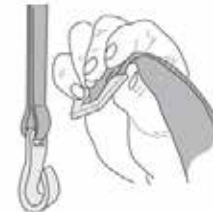
– Altair

Hand control

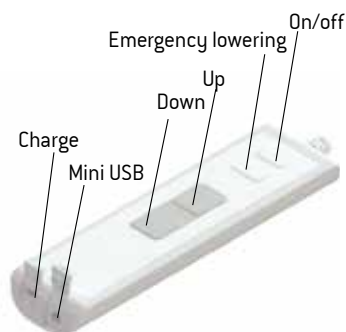




⚠ The covers for the USB input and the charging input must be closed for the hand control's water protection (IPX5) to be valid.

Suspending the lift from the rail

	
<p>Place the lift on a suitable surface. Feed out enough of the belt by pulling gently on the hook on the lift belt (2) while at the same time keeping the Down button on the hand control or on the control panel of the lift pressed in.</p>	<p>Hook onto the hook on the distance strap hanging from the rail roller diagonally from above (see fig) Switch on at the mains and the lift is ready for use.</p>

The telescopic hanger bar

			<p>⚠ Important! Do not hold on to the telescopic bar or the hooks when performing a lift.</p>
<p>The hooks on the telescopic bar have three adjustable positions (40, 50 and 60 cm). To change position, turn the hooks 1/4 turn while pushing or pulling the hooks to the desired position. The hooks will automatically fall into place in the position, and so it is necessary to repeat the turn (1/4 turn) when passing the middle position (position 2) (See fig).</p>			



IMPORTANT! This quick reference guide does not replace the user manual which can be downloaded at www.humancaregroup.com

AUSTRALIA

Unit 9, 271 Wells Road
Chelsea Heights, Victoria 3196
Phone:- +61 3 8773 1111
Fax:- +61 3 9773 4008
Email: info.au@humancaregroup.com

CANADA

10-155 Colonnade Road
Ottawa, ON K2E 7K1
Phone:- 613.723.6734
Fax:- 613.723.1058
Email: info.ca@humancaregroup.com

SWEDEN

Årstaängsvägen 21C
117 43 Stockholm
Phone:- +46 8 665 35 00
Fax:- +46 8 665 35 10
Email: info.se@humancaregroup.com

UNITED STATES

4210 S. Industrial Drive, Suite 160
Austin, TX 78744
Phone:- 512.476.7199
Fax:- 512.476.7190
Email: info.us@humancaregroup.com